

Flexible Stabilizer Instrument: Flexible Tube Replacement

Stabilizer Handle Assembly: CT-0188 & CT-0197

Replacement Flexible Tubes: CT-0186 & CT-0195

INSTRUCTIONS:

1. Loosen the Collet on the Handle Assembly.
Hold the *Top Knob* [A] firmly with one hand and turn the *Bottom Knob* [B] counterclockwise with the other hand to loosen the *Collet* [D].
2. Pull the used *Flexible Tube* [C] out from the *Collet* [D] and discard.
3. Insert the slotted end of a new, replacement *Flexible Tube* into the loosened *Collet* of the Handle.
4. Rotate the tube until the slots seat fully into the *Collet* and the *Laser Mark* [E] on the tube is immediately adjacent the *Collet*.
5. Tighten the Collet on the Handle Assembly.
Hold the *Top Knob* firmly with one hand and tighten (turn clockwise) the *Bottom Knob* with the other hand until tube is secure.

