Description

IlluminOss offers sets with a variety of standard, reusable surgical instruments to assist orthopedic surgeons with the implantation procedure for the IlluminOss Photodynamic Bone Stabilization System. The intended purpose of the IlluminOss Reusable Instruments is for use in conjunction with the IlluminOss implants, for obtaining access to the bone, the preparation of the bone canal, the delivery of the implant, and if needed the removal of the implant. The IlluminOss Reusable Surgical Instruments are used only in conjunction with the IlluminOss Photodynamic Bone Stabilization System, and therefore the indications, contraindications, intended patient population, and intended users that govern their use are those of the implant, the IlluminOss Photodynamic Bone Stabilization System.

These reusable surgical instruments must be cleaned and sterilized before use. The information provided is for the reusable surgical instruments only, and does not apply to the Photodynamic Bone Stabilization System catheters (implant), or sterile disposable ancillary devices. Refer to the implant Instructions for Use for the implant indications, contraindications, warnings, precautions and risks (ref: 900535 found on www.illuminoss.com).

Instrument Sterility, Cleaning, and Decontamination

The Surgical Instrument Sets and their instruments are supplied non-sterile and must be sterilized before use unless otherwise indicated. Thoroughly clean instruments prior to initial sterilization. When handling sharp instruments, use extreme caution to avoid injury. Consult with an infection control practitioner to develop and verify safety procedures appropriate for all levels of direct instrument contact. Instruments returned to IlluminOss or its distributors should be cleaned and sterilized prior to shipment. ANSI/ AAMI ST35 "Safe Handling and Biological Decontamination of Reusable Medical Devices in Health Care Facilities and in Nonclinical Settings" provides guidelines for return, or contact IlluminOss or your distributor for further instruction.

Point of Use:	<p>Clean instruments as soon as possible, after use. Do not allow blood to dry on the instruments. Wash all instruments whether or not they were used or inadvertently came into contact with blood or saline solution.</p> <ul style="list-style-type: none"> If cleaning must be delayed, place groups of instruments in a covered container with appropriate detergent or enzymatic solution to delay drying.
Cleaning:	<p>Manual Cleaning/Disinfection Instructions:</p> <ul style="list-style-type: none"> Immediately after use in the surgical procedure and instrument disassembly, remove as much visible debris as possible from each instrument and its components using moist clean gauze pads or wipes. Instruments should be soaked immediately after use. If the instruments cannot be soaked immediately, they should be wrapped in a clean moist towel. Soiled instruments must be kept moist to prevent the soil from drying. Prepare an enzymatic cleaning solution with warm clean tap water (35-43°C) per the instructions of the solution manufacturer. Perform an enzymatic soak for a minimum of 20 minutes making sure the instruments are completely immersed in the solution. Actuate all moving mechanisms a minimum of 5 times (5x) to ensure the enzymatic solution contacts all parts. While in the soak solution, use a soft brush or clean cloth to manually remove all the exterior soil. Use appropriately sized brushes to thoroughly clean the entire length of any lumens including all other challenging design features such as holes, hinge/mated surfaces, crevices, serrations, etc. Scrub each design feature a minimum of 5 times (5x). After the enzymatic soak, rinse instruments thoroughly with clean warm tap water (35-43°C), taking care to flush all lumens and crevices. Rinse thoroughly for at least one minute under a continuous stream of clean water (at a minimum flow rate of 1 Liter/minute) ensuring all surfaces are rinsed and the instruments visibly clean. If rinse water does not run clear or there are any visual signs of soil, repeat cleaning process. Prepare a fresh enzymatic cleaning solution with warm clean tap water (35-43°C) per the instructions of the solution manufacturer. This solution will be used in the ultrasonic bath. The instruments should be fully immersed in the cleaning solution and all mechanisms actuated a minimum of 5 times (5x). Run the ultrasonic cleaning unit for a minimum of 15 minutes. Place the instruments in clean warm (35-43°C) deionized (DI) or reverse osmosis (RO) purified water and again scrub the instruments, ensuring all the surfaces are scrubbed, including the lumens, interfaces, and crevices. Scrub each design feature a minimum of 5 times (5x). Rinse each instrument again for at least one minute under clean warm DI or RO (35-43°C) running water (at a minimum flow rate of 1 Liter/minute), including all design features. Ensure that all surfaces are free of visible contamination or any traces of the cleaning solution (e.g. foam) and the fluid runs clear. Dry the instruments with a clean soft cloth and use clean compressed air to dry the lumens, holes, and interfaces. Perform a visual inspection on the instruments and verify that they are clean, dry, and in proper working order. If the instruments are not visually clean, repeat the cleaning process.

	<p>Automated Cleaning/Disinfection Instructions:</p> <ul style="list-style-type: none"> Automated washer/disinfector systems are not recommended as the sole cleaning method for surgical instruments. Orthopaedic instruments should be cleaned following the manual or combination manual/automated cleaning procedure except where specifically indicated. Simple instruments without multiple components, lumens/cannulations, blind holes, mated surfaces, connectors and internal mechanisms or other complex features may be successfully cleaned and disinfected using a typical washer/disinfector cycle for surgical instruments as outlined below. Devices should be thoroughly inspected prior to sterilization to ensure effective cleaning. <p>Typical European Automated Washer/Disinfector Cycle for Surgical Instruments</p> <table border="1"> <thead> <tr> <th>Step</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>5 min pre-rinse with cold tap water</td> </tr> <tr> <td>2</td> <td>10 min wash with 0,7% cleaning agent at 55°C</td> </tr> <tr> <td>3</td> <td>1 min rinse with cold tap water</td> </tr> <tr> <td>4</td> <td>Disinfection at 93°C with hot purified water until A₀ 3000 is reached (approx. 10 min)</td> </tr> <tr> <td>5</td> <td>40 min hot air drying at 110°C</td> </tr> </tbody> </table> <p>Note: The washer/disinfector manufacturer's instructions should be strictly adhered to. Use only cleaning agents recommended for the specific type of automated washer/disinfector. A washer/disinfector with approved efficacy (e.g. CE mark, FDA approval, and validation according to ISO 15883) should be used.</p>	Step	Description	1	5 min pre-rinse with cold tap water	2	10 min wash with 0,7% cleaning agent at 55°C	3	1 min rinse with cold tap water	4	Disinfection at 93°C with hot purified water until A ₀ 3000 is reached (approx. 10 min)	5	40 min hot air drying at 110°C
Step	Description												
1	5 min pre-rinse with cold tap water												
2	10 min wash with 0,7% cleaning agent at 55°C												
3	1 min rinse with cold tap water												
4	Disinfection at 93°C with hot purified water until A ₀ 3000 is reached (approx. 10 min)												
5	40 min hot air drying at 110°C												
Sterilization:	<ul style="list-style-type: none"> All instruments should be placed in their designated instrument trays and the tray wrapped with an approved CSR sterilization prior to sterilization. Effective sterilization is predicated on thorough cleaning and drying processes. Failure to do so may compromise the sterilization process and render the processed instruments unsuitable for clinical use. Wrap Instrument Case in an approved CSR sterilization wrap according to local procedures using techniques such as those outlined in AAMI ST79. Sterilize in an approved sterilizer. Do not stack trays during sterilization. Sterilize Instruments using PreVac steam autoclave or Gravity steam autoclave: <ol style="list-style-type: none"> For PreVac steam autoclave, sterilize instruments for a minimum of 4 minutes at 270°F (132°C) and at cycle completion, dry for a minimum of 20 minutes at 270°F (132°C). <p style="text-align: center;">OR</p> <ol style="list-style-type: none"> For PreVac steam autoclave, sterilize instruments for a minimum of 3 minutes at 134°C (273°F) and at cycle completion, dry for a minimum of 16 minutes at 134°C (273°F). <p style="text-align: center;">OR</p> <ol style="list-style-type: none"> For Gravity steam autoclave, sterilize instruments for a minimum of 15 minutes at 270°F (132°C) and at cycle completion, dry for a minimum of 15 minutes at 270°F (132°C). 												
Inspection:	<ul style="list-style-type: none"> Inspect the instrument case and instruments for wear or damage upon receipt and after each use and cleaning. Cutting edges should have continuous edge and be free of nicks. Jaws and teeth should align and engage properly. Movable parts should move smoothly without excessive play and locking mechanisms should fasten securely. Do not use worn or damaged instruments. Contact IlluminOss immediately for return, repair, or replacement of damaged or worn instruments. Instruments with dents, burs, signs of corrosion or spotting, or raised surfaces which could cause damage to surgical gloves should be set aside for repair service or returned to IlluminOss. Incompletely cleaned instruments should be re-cleaned. Do not reuse or attempt to clean/sterilize single use instruments. These instructions are applicable to the IlluminOss Reusable Surgical Instruments, only. 												

SY-3000 SY-4000 SY-5000 SY-9100
 CT-0239 CT-0241 CT-0242 CT-0186 CT-0188 CT-0196 05-0901 CT-0129 CT-0131 CT-0238 CT-0278
 CT-0235 CT-0268 CT-0037 CT-0038 CT-0039 CT-0040 CT-0041 CT-0042 CT-0151 CT-0146 CT-0143 CT-0161
 CT-0156 CT-0153 CT-0162 CT-0163 CT-0199 CT-0127 CT-0195 CT-0197 CT-0301 CT-0267 CT-0266 CT-0206
 CT-0207 CT-0231 CT-0232 CT-0233 CT-0225 CT-0226 CT-0227 CT-0228 CT-0229 CT-0237 CT-0270 CT-0216
 CT-0305 CT-0306

ILLUMINOSS MEDICAL INC.
 993 Waterman Avenue
 East Providence, RI 02914
 U.S.A.
 +1 (401) 714-0008

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