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IlluminOss Medical GmbH  
Mühlenhof 7-9  
40721 Hilden

**Instructions for Preparing IlluminOss Instrument Sets**  
**SY-3000, SY-4000, SY-5000, SY-9100**  
(applies in combination with the surgical technique guide and IFU 900015)

We hereby confirm that the IlluminOss instrument is suitable for additional mechanical/thermal cleaning/disinfection after manual cleaning, with a mildly alkaline cleaner (pH 11), following the cleaning steps below:

1. Manual Cleaning (per 900015)
2. Pre-cleaning 5 min at 20°C
3. Cleaning 10 min at 55°C
4. Cleaning rinse 1 min
5. Disinfection 10 min at 93°C (reaching AO3000 value)
6. Drying 40 min at 110°C

The instruments should then undergo steam-sterilization at 134°C, with fractioned pre-vacuum and a holding time of 3 min (2 alternative methods are outlined in 900015).

We outline our manual cleaning requirements in 900015 (see [www.illuminoss.com](http://www.illuminoss.com)).

Note: Strictly follow the instructions of the cleaning/disinfector manufacturer. Use only the detergents recommended for each type of cleaner/disinfector. Only cleaning/disinfection equipment with certified efficacy should be used (e.g. CE marking, FDA approval and validation according to ISO 15883).

IlluminOss Medical GmbH  
Mühlenhof 7-9  
40721 Hilden