**Instructions for Preparing IlluminOss Instrument Sets**  
**SY-3000, SY-4000, SY-5000, SY-9100**  
(applies in combination with the surgical technique guide and IFU 900015)

We hereby confirm that the IlluminOss instrument is suitable for additional cleaning **after** manual cleaning, either with a neutral pH / enzymatic detergent, or using an alkaline pH / non-enzymatic detergent, following the cleaning steps below:

- |                         |                                    |
|-------------------------|------------------------------------|
| 1. Manual Cleaning      | (per 900015)                       |
| 2. Pre-wash             | 5 min cold tap water               |
| 3. Cleaning             | 10 min detergent wash at 60°C      |
| 4. Cleaning rinse       | 1 min with hot utility water       |
| 5. Thermal Disinfection | A0 = 3000 at 93°C (approx. 10 min) |
| 6. Drying               | 20 min at 110°C                    |

The instruments should then undergo steam-sterilization at 134°C, with fractioned pre-vacuum and a holding time of 3 min (2 alternative methods are outlined in 900015).

We outline our manual cleaning requirements in 900015 (see [www.illuminoss.com](http://www.illuminoss.com)).

Note: Strictly follow the instructions of the cleaning/disinfector manufacturer. Use only the detergents recommended for each type of cleaner/disinfector. Only cleaning/disinfection equipment with certified efficacy should be used (e.g. CE marking, FDA approval and validation according to ISO 15883).

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IlluminOss Medical GmbH